Magnification

Compact 10 HD

Compact 10 HD Speech

User manual
Gebruikershandleiding
Manuel d'utilisation

Manuale utente
Manual del usuario
Bedienungsanleitung

Optelec

Contents

1. INTRODUCTION..... 15
 ABOUT THIS MANUAL 15
 WHAT’S IN THE BOX? 16

2. GETTING STARTED 17
 2.1. CHARGING THE BATTERY 18
 Battery warning..... 19
 2.2. POWER ON AND OFF 19
 Standby 19
 Sleep and resume..... 20
 2.3. FOLDING AND UNFOLDING THE COMPACT 10 HD..... 21
 2.4. CLEANING THE CAMERAS 22
 2.5. BUTTONS AND CONNECTIONS 23

3. CAMERAS..... 24
 3.1. MAGNIFIER CAMERA 24
 3.2. FULL PAGE CAMERA 24
 3.3. OVERVIEW CAMERA 25

4. USING THE COMPACT 10 HD SPEECH’S TEXT-TO-SPEECH & OCR..... 26

5. EASY AND ADVANCED MODE 28
 5.1 FEATURES IN EASY MODE 30
 Adjusting the magnification 30
 Selecting a viewing mode 30
 5.2. FEATURES IN ADVANCED MODE 31
 Time and date..... 31
 MENU..... 32

Voice (Compact 10 HD Speech only).....	34
Line On/Off	35
Battery status.....	35
Save	35
Open.....	36
Adjusting the brightness.....	37
SETTINGS	38
Color	39
View (Compact 10 HD Speech only)	40
Miracast: connecting wirelessly to a monitor or TV	40
Bluetooth (the Compact 10 HD Speech only).....	42
Line.....	43
Buttons	43
Theme	44
Columns (the Compact 10 HD Speech only).....	44
Lights.....	44
Sound.....	44
Standby	45
Language (the Compact 10 HD only)	45
Restore.....	45
APPENDIX A: TECHNICAL INFORMATION.....	46
APPENDIX B: SAFETY AND REGULATORY NOTICES	47

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User Manual

Version 1.0

© 2019 Optelec, the Netherlands
All rights reserved

Optelec
P.O. Box 399
2990 AJ Barendrecht
The Netherlands

Phone: +31 (0)88 678 34 44
E-mail: info@optelec.nl
Internet: www.optelec.com

1. Introduction

Congratulations on choosing the Compact 10 HD or the Compact 10 HD Speech!

With its large 10" screen, customizable features and intuitive operation, the Compact 10 HD offers all benefits of a desktop video magnifier, but in a much smaller, foldable, and portable design. With the arm unfolded, you can easily look at photos, write your signature, view medicine packaging, and even read round cans, without being restricted in your movements. With the Compact 10 HD Speech, the swing-out arm allows you to scan and read full pages of text.

If you have questions or suggestions concerning the use of this product, please contact your distributor or Optelec's headquarters, using the contact information on the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your the Compact 10 HD!

About this manual

At Optelec, we are constantly improving our products and their functionality. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on www.optelec.com at the Support section.

This manual will familiarize you with the features and basic operation of the Compact 10 HD and the Compact 10 HD Speech. Please read this manual carefully before using your device.

Unless otherwise indicated, all references to the Compact 10 HD will also apply for the Compact 10 HD Speech. Functionality specifically applicable for the Compact 10 HD Speech are marked with the Speech logo.

What's in the box?

The Compact 10 HD packaging contains the following:

- The Compact 10 HD or the Compact 10 HD Speech video magnifier
- A protective pouch
- A USB power supply with USB-C cable
- 4 country-specific power plugs (EU, US, UK, and AU)
- A screen cleaning cloth
- This user manual

If any of these items are missing from your package, please contact your distributor.

2. Getting started

The Compact 10 HD is placed correctly in front of you when the screen is facing up and towards you, with the Optelec logo below the screen.

2.1. Charging the battery

The Compact 10 HD is delivered with a power supply and four country-specific power plugs in EU, US, UK, and AU types. Select the power plug that corresponds to your country and plug it into the power supply.

To charge the Compact 10 HD, connect the supplied USB cable to the power supply and the blue USB connector located on the left side of the unit. Plug the other end of the power supply into a power outlet.

Battery full

Battery charging

Battery almost empty

Charging the Compact 10 HD takes approximately 3 hours. A fully charged the Compact 10 HD can operate up to a maximum of 4 hours, depending on the power settings and usage. For safety reasons, please only use the power supply and USB cable that was delivered with the Compact 10 HD to charge the unit's battery.

Battery warning

The Compact 10 HD will briefly show an empty battery warning icon to indicate that the battery needs to be charged. When this occurs, make sure to charge your unit. When the battery is empty, the unit will automatically switch off.

If the Compact 10 HD does not switch on, please make sure it is charged.

2.2. Power on and off

To switch on the Compact 10 HD for the first time, the unit needs to be charged. Once charged, press the orange On / Off switch. To switch off the Compact 10 HD, press the orange button for three seconds again. When switching on the Compact 10 HD, it will take approximately 25 seconds before an image displays on screen.

Standby

The Compact 10 HD can automatically go into standby mode if no button is pressed or no movement is detected. Pressing the orange on / off button will switch it back on. The standby time can be changed or disabled in the Compact 10 HD menu. Please refer to chapter 5.2 on how to change these settings.

Sleep and resume

When the Compact 10 HD is switched on, a short press of the orange On / Off button will place the unit into sleep mode. A short press of the orange On / Off button again will resume the Compact 10 HD operation in the mode in which it was put to sleep. When resuming from sleep, the Compact 10 HD will take approximately 1 second before an image displays on screen.

When the Compact 10 HD is in sleep mode, the unit can also resume by opening unfolding the Compact 10 HD. Closing the stand will automatically put it back into sleep mode.

2.3. Folding and Unfolding the Compact 10 HD

When unfolding the Compact 10 HD, the screen will be set at an ergonomic angle for reading, and the magnifier camera will be activated.

To **open** the Compact 10 HD, follow these steps:

1. Place the Compact 10 HD in front of you so that the screen is facing up, and the Optelec logo is below the screen.
2. Grab and hold the Compact 10 HD in an upright position. Your fingers hold the feet of the stand while your thumbs hold the side of the Compact 10 HD.
3. With your fingers, grab the edge of the feet of the stand within the recess at the back of the unit.
4. Unfold the feet of the stand until they click in position.

To **close** the Compact 10 HD, follow these steps:

1. Place the Compact 10 HD in front of you so that the screen is facing towards you.
2. Hold the Compact 10 HD at the sides and place your fingers on the supports between the feet and the unit.
3. Gently press the supports of the feet towards you so they bend.
4. Tilt the screen down until it clicks in position.

2.4. Cleaning the cameras


For an optimal quality image, it is recommended to ensure that the camera lens and object light window are kept clean. A dirty camera window will result in a fuzzy image and may cause the auto-focus camera to incorrectly focus. To clean the cameras, use the provided cleaning cloth or another soft lens cleaning cloth.

Note: Do not use water or cleaning agents to clean the camera window.

2.5. Buttons and Connections

The Compact 10 HD has the following buttons and connections at the left side of the product: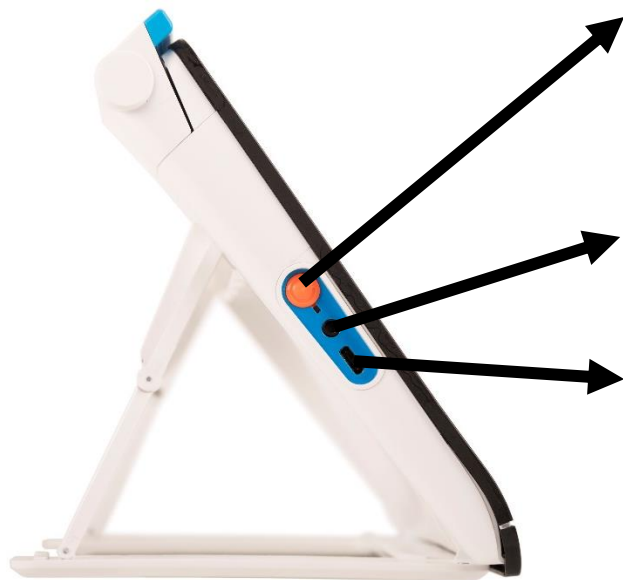

On / Off button

A short press of the orange On / Off button will place the Compact 10 HD into sleep mode or resume the unit. A 3 second press of this button switches the unit off or back on again.

Audio jack

Use the audio jack to connect a headset to the Compact 10 HD.

USB-C connector

Use the USB-C connector to charge the Compact 10 HD. The supplied USB cable connects to this USB port and power supply. Use only the power supply that comes with the Compact 10 HD to power and charge the unit.

3. Cameras

The Compact 10 HD is equipped with three cameras, each serving a specific task or situation.

3.1. Magnifier camera

The Magnifier camera is automatically used when the Compact 10 HD is in the open position with the stand unfolded. The Magnifier camera is used for reading on a desk or table. The Compact 10 HD is placed on top of the reading material.

3.2. Full Page camera

The Full Page camera is automatically used when the swing-out camera arm is opened. The Full Page camera can be used to view photos and objects more comfortably, or to write your signature, without being restricted in your movements.

By tapping the screen, the focus can be locked for writing or viewing objects at a fixed distance.

If you own a the Compact 10 HD Speech, the full page camera is also used to scan and read full page documents. Place the document you want to read below the camera, aligning the bottom left corner of the document with the blue indication at the right side of the base of the Compact 10 HD Speech.

3.3. Overview camera

The Overview camera is automatically used when you power on the Compact 10 HD while the Compact 10 HD stand is closed. To switch from Magnifier camera to Overview camera, close the stand and press the On / Off button. The overview camera is optimized to look at objects at a short distance, such as time tables.

4. Using the Compact 10 HD Speech's text-to-speech & OCR

If you own a the Compact 10 HD Speech, taking a snapshot activates the Speech functionality. The Compact 10 HD Speech will extract the text from the image and will read the text aloud.

Activating the speech

To take a snapshot and activate the Speech:

- Position the text or part of the text in the camera view and press the snapshot button. This can be done using the Magnifier camera, or the Full Page camera. When taking a snapshot of a full page, make sure you align the bottom left corner of the document with the blue indicator at the right side of the stand of the Compact 10 HD Speech. For an optimal result, make sure there is enough light in the environment, prevent shadows and limit the text to only the part of the document you would like to read. If the text in view gets too small, the recognition will lose accuracy and might fail.
- A progress bar will be shown. the Compact 10 HD Speech will start reading after the recognition process has been finished.
- Use the Play/Pause button to start or stop reading.
- Tap and hold (long press) on the text to start reading at the word or text section you point at.

- Press the “Mode” button to switch between the various high contrast options and full color mode.
- If no text is found, a “No text found” message will appear on pressing the “Play” button.

Adjusting the volume and reading speed

To adjust the volume or reading speed, activate the button bar with a single tap. Tap the Audio button to display the “Volume” and “Speed” sliders. The “Volume” and “Speed” sliders are only present in the snapshot view.

Adjusting language and voice

To adjust the language or voice in Easy Mode, press and hold the “Play” button to go to the Voice menu.

To adjust the language or voice in Advanced Mode, press the “Menu” button and then press the “Voice” menu.

Please refer to Chapter Chapter 5.2, Voice for more information on how to navigate the “Voice” menu.

5. Easy and Advanced mode

Upon first using the unit, it will start up in Easy mode. To switch between Easy and Advanced mode, press and hold the Mode button and tap Easy or Advanced.


In Easy mode, you have access to the main features:

- Adjusting the magnification
- Selecting a viewing mode
- Taking a snapshot
- If you own a the Compact 10 HD Speech,
 - The text in the snapshot will be read out aloud.
 - The reading speed and volume can be adjusted.
 - Changing the language and reading voice
- Switching between the Magnifier and Overview camera


In Advanced mode, you have the same functions as in Easy mode, plus some additional features:

- Time and date
- Access to the menu with the options to:
 - Save and open snapshots
 - Changing the language and reading voice
 - Changing color modes
 - Enabling the Reading Line
 - Bluetooth audio and Miracast

- Other advanced settings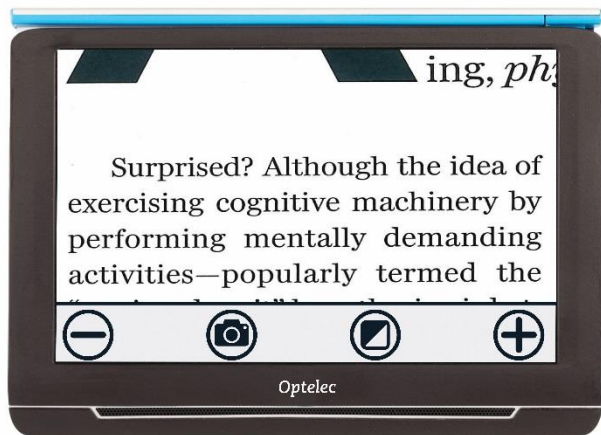

Easy Mode

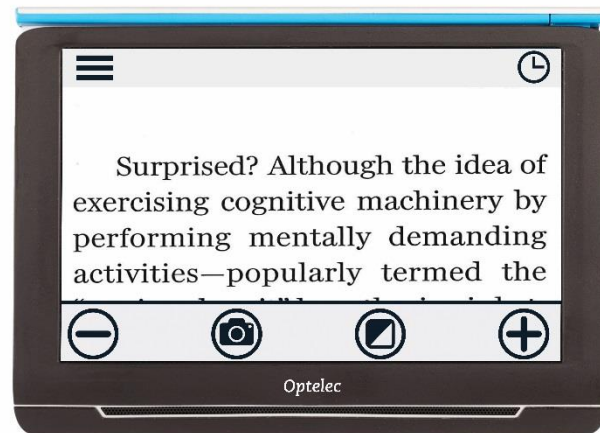Advanced Mode

5.1 Features in Easy mode

Adjusting the magnification

To change the magnification, simply tap the screen. The button bar will be shown at the bottom of the screen or at the right side of the screen, depending on the chosen button bar position. The button bar position can be selected in the menu. See chapter 7 for more information on how to change the button bar position. In the button bar tap the **+** button to increase or the **-** button to decrease the magnification. Alternatively, use the pinch-to-zoom gesture to change the magnification. Tap the screen to hide the button bar. The button bar will automatically hide when the screen is not touched for 10 seconds.

Selecting a viewing mode


Tapping the Mode button in the middle of the button bar allows you to select one of the following viewing modes:

- **Photo mode:** Viewing full color texts and photos
- **Negative mode:** Reading in high contrast, white on black
- **Positive mode:** Reading in high contrast, black on white
- High contrast **color combination 1**
- High contrast **color combination 2**

5.2. Features in Advanced mode

Time and date

To open the clock, tap the screen and tap on the “Clock” icon at the right edge of the screen.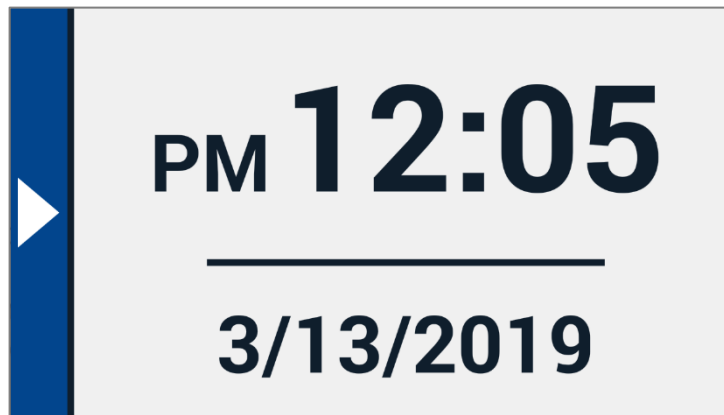


Use the “Back bar” to exit the time and date screen.

To set the time and date, tap the time or date. To change a value, tap on the field you would like to change and tap on the up or down symbol to change the value. Tap on the “Back bar” to exit the set time and date screen.

Menu

Opening and closing the menu

To open the menu, tap on the screen and tap on the “Menu” icon at the left side of the screen. This will open the main menu. To close the menu, tap on the “Back bar”.

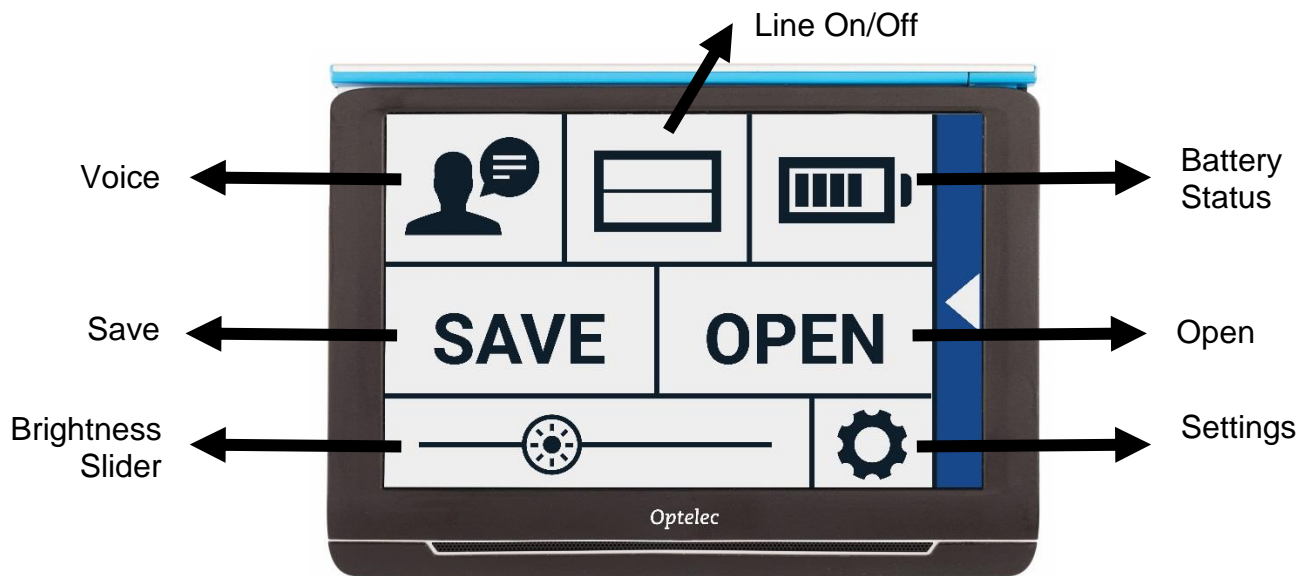The exact items in the menu will vary, depending on if you own a the Compact 10 HD or 10 HD Speech. To activate a button, tap the button once. Activating a button will open a new menu screen or list. To go back to the main menu, tap on the “Back bar”. To adjust the slider, tap and hold the slider button while moving it to the right or left.

The main menu contains the following options:

- Voice (Compact 10 HD Speech only)
- Line On/Off
- Battery status
- Save
- Open
- Brightness slider
- Settings

Voice (Compact 10 HD Speech only)

To select a voice, language and menu language, or to add or remove a voice or language, open the menu and tap the “Voice” button.

Selecting a voice, language and menu language

This menu item lets you select your reading voice and language. To select another reading voice, tap the voice icon in the top left corner of the main menu to open the list with preselected voices. Each voice represents a language. Tap on a voice to select the voice. If adjusting the reading voice results in a change of language, the menus will automatically change to that language.

Adding or removing a voice or language

To add or remove a voice or language, tap the voice icon in the top left corner of the main menu. In the list with preselected voices, tap on the “Add/Remove” button. To add or remove a reading voice, select the language from the list by tapping on the language. This will display a list with available voices for the selected language. To enable a reading voice, select it from the list by tapping on it. A checkmark in front of the voice name will indicate if it is enabled. When the voice is enabled, it will be available under the “Voice” button in the main menu. By deselecting a voice, the voice will be removed from the available reading voices. The languages of which a reading voice is selected, will appear at the top of the list with languages, and are marked with a checkmark.

Adjusting language and voice in Easy Mode

In Easy mode, you cannot access the Menu. To adjust the language or voice in Easy Mode, press and hold the “Play” button to go directly to the Voice menu and follow the steps described above.

Line On/Off

The Compact 10 HD can display a reading line to help you position the Compact 10 HD straight on the document. To enable the reading line, tap the “Line” button. Tap this button again to hide the reading line.

Battery status

The battery status is shown in the top right corner of the screen. If you tap the battery icon, a new screen will be opened showing a magnified battery icon and battery status, the “About” button and the “Back arrow” to return to the main menu. The lightning symbol indicates that the battery is charging.

About

Tap on the “About” button in the battery screen to see the application version.

Save


To save a snapshot, open the menu and tap the “Save” button.

- Make a snapshot using the snapshot button
- Open the menu
- Press the save button to save the snapshot
- The menu will close and the snapshot will be shown


If there is no snapshot to save, or if the snapshot has already been saved, the “Save” button will be hidden.

Open

To open a snapshot, open the menu and tap on the “Open” button. A list of thumbnails of the saved snapshots will be shown, showing the most recently saved snapshot first. If there are more snapshots than fit on the screen, swipe up and down to scroll through the list. Tap on the snapshot thumbnail to open it.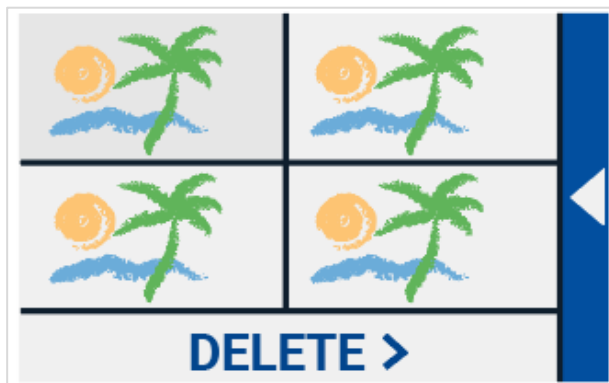

Open to view saved snapshots

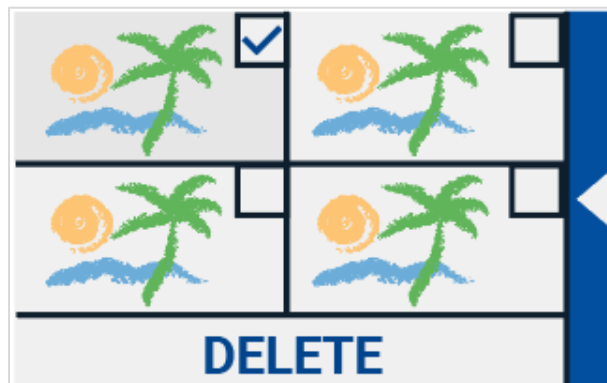Delete snapshots

Delete

To delete a snapshot, open the menu and tap on the “Open” button. In the open screen, tap on the “Delete” button. A list of thumbnails of the saved snapshots will be shown, showing the most recently saved snapshot first. Each snapshot has a checkbox at the top right corner. To select a snapshot, check the checkbox. Tap the “Delete” button to delete the selected snapshots. To delete all snapshots, tap the “Delete” button without selecting a snapshot. You will be prompted to confirm to delete all snapshots. To delete all snapshots, select “Yes”. To cancel deleting all snapshots, select “No”. Both choices will return you to the list of snapshots. Tap the “Back bar” to return to the “Open” screen. Tap the “Back bar” again to return to the main menu.


Adjusting the brightness

Use the slider with the sun icon to increase or decrease the brightness of the screen.

Settings

Tap on the “Settings” button to enter the Settings menu. The “Settings” button is the button with the gear icon at the bottom right. The Settings menu contains the following buttons:

- Color
- View (Compact 10 HD Speech only)
- Miracast
- Bluetooth (Compact 10 HD Speech only)
- Line
- Buttons
- Theme
- Columns (Compact 10 HD Speech only)
- Lights
- Sound
- Standby
- Language (Compact 10 HD only)
- Restore

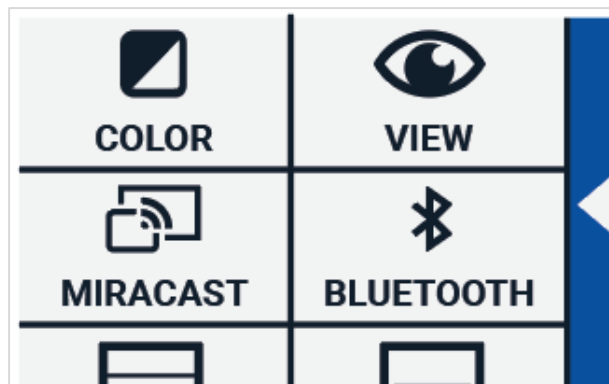Color

In the settings menu, tap the “Color” button to select the preferred high contrast colors. The Compact 10 HD supports four high contrast modes. For each of these four modes, the foreground and background colors can be selected. Three of the high contrast modes can be disabled. One high contrast mode is required.

To change the foreground and background colors:

1. Tap on the mode number you would like to change. Each number represents a mode.
2. To select new high contrast colors for the foreground and background, tap on the up arrow head to move up or the down arrow head to move down within the list of color combinations.
3. High contrast modes 2, 3 and 4 can be disabled by selecting the “X” symbol.
4. Tap on the “Back bar” to return to the settings menu.

View (Compact 10 HD Speech only)

In the settings menu, tap the “View” button to cycle through the different viewing modes.


Press the Mode button to cycle through the different viewing modes Original, Line and Column.

Press the Highlight button to cycle through the different highlight modes: Rectangle and Underline.

Miracast: connecting wirelessly to a monitor or TV

Miracast is a standard for a wireless connection to a monitor. It can be described as HDMI over Wi-Fi", replacing the cable from the Compact 10 HD to the monitor or TV. To be able to use the Miracast function of the Compact 10 HD, your monitor or TV needs to support Miracast. Please refer to your monitor or TV manual and documentation to verify if your device supports the Miracast feature.
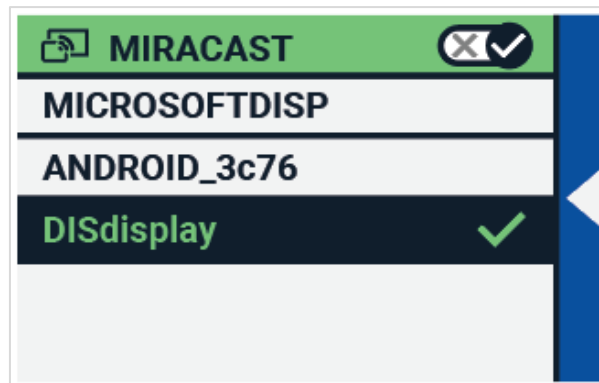


To connect your the Compact 10 HD to a monitor or TV via Miracast, tap the “Miracast” button and Switch Miracast on. A list with available devices will be displayed. Click on a device to connect. It could take up to a minute to establish the connection. Tap on the device again to disconnect. When a Miracast connection is established, both the audio and video will be streamed to the TV or monitor. To guarantee a good connection, the Compact 10 HD needs to be within one meter or three feet of the monitor or TV. Depending on the connection, a short delay of the video and / or audio could be noticed.

Bluetooth (the Compact 10 HD Speech only)

To connect a Bluetooth speaker or headphone, tap the “Bluetooth” button and switch Bluetooth on.

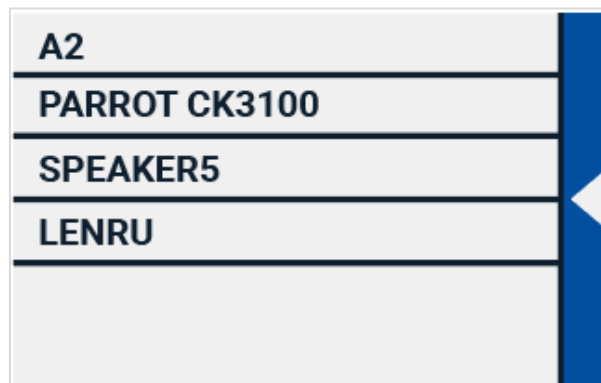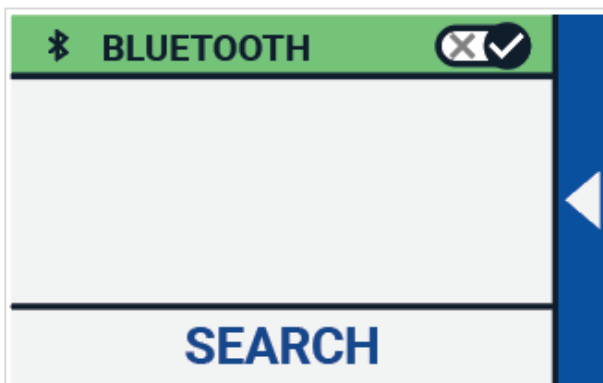A list with paired devices is being displayed.

If no speaker or headphone is being paired, tap the “Search” button to search for available devices. Before starting the Bluetooth search, make sure the speaker or headphone is switched on and is set to discoverable. Please refer to the speaker’s or headphone’s manual on how to switch the device to Bluetooth discoverable mode. To pair the speaker or headphone, tap on the device name

and if needed enter the pin code. Refer to the speaker's or headphone's manual for the pin code of the device.

When the device is paired, to connect the speaker or headphone, tap the device. You will be asked to connect or forget the device. Tap the "Connect" button to make the connection.

To indicate that a device is connected, the connected device will have a checkmark and will be displayed in blue.

To unpair / forget a device, tap the "Forget" button. The device will be removed from the list with paired devices.

Note: Only Bluetooth version 4.0 or higher headsets and speakers are supported.

Line

This menu item enables you to change the reading line position to one of its six positions; horizontal top, horizontal center, horizontal bottom, vertical left, vertical center, vertical right. By default, the reading line is horizontally set to the centre of the screen.

Buttons

This menu item enables you to change the button bar position to one of two positions: bottom or side. Depending on your preference, you could change the position of the button bar to the bottom or side of the screen. The default setting is "Bottom".

Theme

This menu item enables you to select a light or dark theme for the buttons and menus.

Columns (the Compact 10 HD Speech only)

This menu item allows you to choose if you want the Compact 10 HD Speech to only read full columns, or incomplete columns as well. By default, Read full columns only is selected. Please note that if you select to read incomplete columns, not all characters and words are fully captured, which may result in incoherent text.

Lights

This menu item allows you to enable or disable the lights in Magnifier mode and in Overview mode. The Magnifier lights are located to the left and right of the magnifier camera. By default, the reading lights are enabled for an optimal image quality.

The Overview light is located next to the overview camera. By default, the overview light is disabled.

Sound

This menu item allows you to mute the audio feedback and enable or disable the vibration feedback. By default, sound and vibration are enabled.

Standby

The standby time sets the time after which the Compact 10 HD goes into standby mode to conserve power. Pressing any button will take the unit out of standby mode. By default, the standby time is set to 5 minutes after no button is pressed or no movement of the Compact 10 HD is detected.

Language (the Compact 10 HD only)

This menu item allows you to choose the language of the Compact 10 HD. Tap to select your preferred menu language.

Restore

Tap this menu item if you wish to restore the default settings. You will be prompted if you are sure to restore the defaults. To restore the defaults select “Yes”. To cancel restoring the defaults, select “No”.

Appendix A: Technical information

Magnification	<0.5 – >22 times
Viewing modes	Full color photo mode High contrast, white on black High contrast, black on white High contrast color combination 1 High contrast color combination 2
Focus	Auto-focus
Screen	10" TFT
Screen resolution	1280 x 800
Dimensions	261 x 189 x 36 mm / 10.28 x 7.44 x 1.42 in
Weight	915 grams / 2.02 lb
Battery	Up to maximum 8 hours use 4,5 hours charging time
Li-ion battery	Rechargeable, 3.7V, 7600mAh, 28.12Wh
Power supply type	Shenzhen Fujia Appliance Co., LTD. FJ-SW1260502500UN
Input power rating – Adaptor	100-240V, 50-60Hz, 0.4A
Output power rating – Unit	DC5V-2.5A

Appendix B: Safety and Regulatory Notices

Please read the instructions carefully before setting up your Compact 10 HD. Keep this user manual for future reference.

Intended Use:

The Compact 10 HD is designed in such a way that it can be used in a hospital environment for general purposes such as reviewing medical records. The Compact 10 HD can not be used for surgical procedures in combination with life supporting systems.

Warning: Critical diagnostic decisions should not be based on images displayed by this device.

- Do not expose the Compact 10 HD to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the Compact 10 HD. Contact an Optelec authorized service organization in your area.
- To avoid risk of electrical damage, keep your Compact 10 HD away from fluids and chemicals.
- Handle the Compact 10 HD with care. Rough handling will damage the internal components.
- Do not use the Compact 10 HD near inadequately shielded medical devices.
- Do not try to open the battery compartment or try to remove the battery.
- Do not touch or look into the LED lights.
- Be careful that your fingers and clothing do not get stuck in moving parts (swing-out arm, stand).
- Contact your dealer to service this equipment if necessary. **Do not disassemble your unit** as it will void the warranty.

- Always unplug and switch off the unit before cleaning. Use a dry soft cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.

Use of the Compact 10 HD other than described in this manual will exclude it from warranty conditions.

Operating conditions

Temperature	+10°C to 35°C / 50°F to 95°F
Humidity	< 70%, no condensation
Altitudes	up to 3000 m (9842 feet)
Pressure	700 – 1060 hPa

Storage & transportation conditions

Temperature	+10°C to 40°C / 50°F to 104°F
Humidity	< 95%, no condensation
Altitudes	up to 12192 m (40000 feet)
Pressure	186 – 1060 hPa

Contacting Product Support

If you have questions about operating, installing or configuring your Compact 10 HD, please contact your Optelec distributor or the Optelec office nearest to you. See the last pages of this manual for a complete list of Optelec addresses.

Warranty Conditions

Optelec warrants the Compact 10 HD, effective from the date of delivery, to be free of any defects in material and workmanship.

The warranty is not transferable and does not apply to groups or multi-users. The Compact 10 HD was designed for the individual purchaser to be used at home, school and work. Optelec reserves the right to repair or replace any purchased Compact 10 HD with a similar or improved product.

In no event shall Optelec or its suppliers be held liable for any indirect or consequential damages. The original user's remedies are limited to replacement of Compact 10 HD modules. This warranty is valid only when serviced in the country of original purchase, and with intact seals. For additional warranty claims or service during or after warranty period, please contact your Optelec distributor.

Optelec is not responsible for any uses of this device other than those described in this manual.

Accessory: Power

Power supply type: Shenzhen Fujia Appliance Co., LTD. FJ-SW1260502500UN

Input power rating – Adaptor: 100-240V, 50-60Hz, 0.4A

Output power rating – Unit : DC5V-2.5A

Guideline:

- Input rating for Compact 10 HD: 5Vdc. 3A
- Protection against electric shock: Class I
- Protection against harmful ingress of water or particulate matter: IPX0
- The Compact 10 HD is intended to be routinely cleaned and disinfected by the user with a soft cloth after use.

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on February 13, 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure the safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

WARNING: The use of accessories and cables other than those specified and sold by the manufacturer of the Compact 10 HD, as replacement parts for components, may result in increased emissions or decreased functioning of the Compact 10 HD.

RoHS Compliance

This product is in compliance with Directive 2011/65/EU of the European Parliament and of the Council of 1 July 2011, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS-II) and its amendments.

This device bears the CE and FCC approval mark.

This device complies with part 15 of the FCC rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Caution for battery pack**CAUTION**

**BATTERY NEEDS TO BE REPLACED BY CERTIFIED SERVICE PERSONNEL.
RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE.
DISPOSE OF USED BATTERIES ACCORDING TO THE REGULATIONS.**

Markings Explanation (on label):

	Do not throw this electronic device into the trash when discarding. To minimize pollution and ensure utmost protection of the global environment, please recycle.
	This side up.
	Fragile.
	Keep dry.

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U.S.A.
Optelec GmbH